ROLLATOR / OCCASIONAL TRANSPORT CHAIR

User Manual

REF RL100

ROLLATOR / OCCASIONAL TRANSPORT CHAIR

ROLLATOR / GELEGENHEITS-TRANSPORTSTUHL

ROLLATOR / INCIDENTELE TRANSPORT STOEL

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User Manual

Instructions

1. Remove the rollator from the carton and check whether the parts and packaging are complete.
2. Place the frame on the flat ground and turn the upper frame upward until it cannot be turned.
3. Corresponding letters (for example, A-A) Install corresponding components
4. Install foot, foot can be turned storage. Note: Foot is forbidden to stand people!
5. Install shopping bags. Weight capacity $\leq 5\text{kg}$

2.

3.

4.

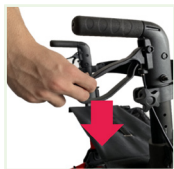
5.

1. Backrest
2. Seat
3. Front wheels
4. Handle
5. Brake
6. Adjustable button
7. Footrest
8. Brake pad
9. Bag
10. Rear wheels

When the implementation process needs to slow down. Pull the left and right brake handles up. You can slow and steady down the speed.

With parking locked, push the brake lever down to the bottom. To unlock, pull the brake handle upward wheel wear will affect the braking effect, please test the braking effect regularly. The brake effect can be improved by adjusting the screw of the brake pad.

Parking rest note:

Be sure to park on a flat and non-skid ground, ensure that the parking is locked in place before sitting down, sit up straight, and do not lean back hard.

Backrest can be turned over, as a simple occasional transport chair.

Note: Seated person push Max. 120 KG!

Folding storage

Caution

This aid is specifically designed for mobility support and should not be used for other activities. Use the aid only as intended.

1. Can be used both as a rollator and a occasional transport chair.
2. Only use on flat and solid ground.
3. Maximum user weight: 120 kg.
4. Regularly check the screws and fasteners on the rollator and re-tighten them if necessary.
5. Only use the bag for transportation, do not overload, the maximum load: 5 kg.
6. Do not expose the product to temperatures above 40°C for a long time, otherwise it will cause overheating of the rollator parts and burn the skin.
7. Make sure that rollator is fully open, the folding bar under the seat cushion is locked in a fixed position, and the handrail is adjusted and secured.
8. Test the braking effect before implementing the rollator. When the wheel is locked, the brake pads on the rear wheel should prevent the rear wheel from turning. When the brake handle is released, the wheel should be able to rotate freely.
9. Be sure to lock the parking seat before sitting down. Do not use the seat when the rollator is located on an inclined or uneven ground.
10. Replace the wheel when the tire is severely worn or damaged.

Product parameters

Total length	75cm (without foot) 102cm (with foot)
Total width	64 cm
Total height	85-97.7 cm (adjustable height grip)
Total weight	8.95 kg
User weight	120kg
Seat cushion width	310 cm
Seat cushion height	55 cm

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Medlogics.com Ltd.