
ROLLATOR

User Manual

REF RL200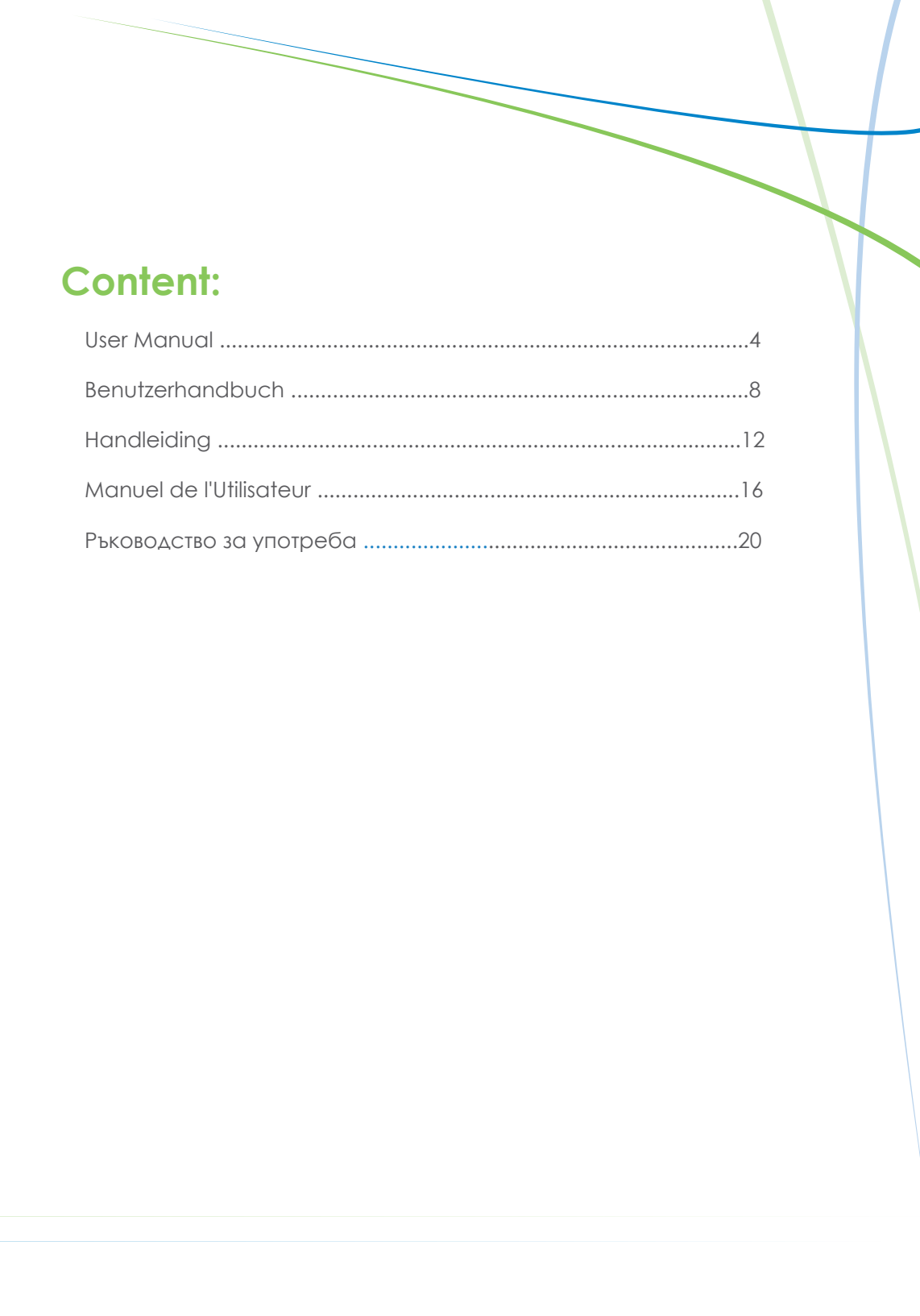



Content:

User Manual	4
Benutzerhandbuch	8
Handleiding	12
Manuel de l'Utilisateur	16
Ръководство за употреба	20

User Manual

Instructions

1. Remove all contents from carton: backrest; handle with tube; Rollator frame, and basket.
2. Open frame and slide padded backrest into frame brackets, making sure snap buttons protrude through holes.
3. Insert handle tube into frame at desired height, lining up holes appropriately.
4. Remove protective plastic covering from tires.
5. Attach basket to front of frame.

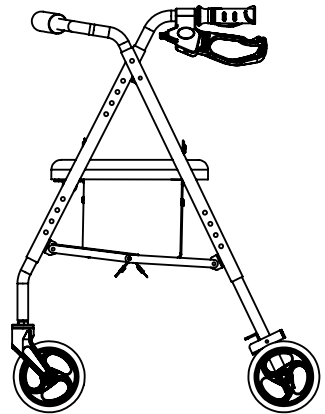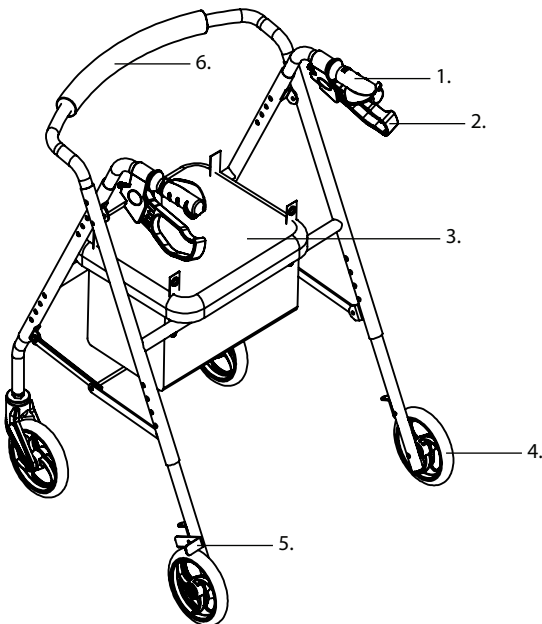1. Handle
2. Brake
3. Seat
4. Rear wheels
5. Brake pad
6. Backrest

Always engage and disengage wheel locks on both sides.

Applying brakes: Squeeze both brake handles to slow or stop.

To lock brakes: Push both brake handles down until they snap into place.


*To release locked brakes: Squeeze both brake handles upward.

To adjust the position of the brake pad, rotate the metal nut in front of the brake handle.

Folding and opening: To fold: Remove the basket and lift the seat cushion. To open: Push down on the seat.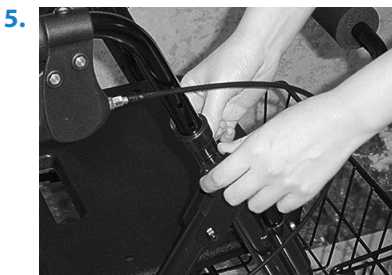

Adjust arm height: Loosen the position knob and remove the bolt. Adjust handle/arm to desired height, then replace bolt and tighten the position knob. Make sure bolt heads are properly countersunk within holes.

Sitting down: When sitting, apply both parking brakes to prevent the Rollator from rolling. Place both feet firmly on the ground and do not lean backward or sideways.

Safety precautions

- Make sure the Rollator is fully opened, seat is down, and the folding bar under the seat is locked in down position.
- Both tightening knobs must be fully tightened and at equal heights. Test Rollator for stability before applying full weight to hand grips.
- Test the loop brakes before operating the Rollator. When wheels are locked, brake bar should prevent wheels from turning. (Refer to points 1 & 2 of “Hand Brake Operation.”) If the wheels turn while the brakes are locked, refer to point 3 of “Hand Brake Operation” for adjustment instructions. If satisfactory locking is not obtained, please take unit back to dealer for inspection. When brakes are released, the wheels should turn freely.
- Do not leave your Rollator unattended without locking its brakes.
- Do not use your Rollator as a seat in any type of motor vehicle.
- Always place brakes in “lock” position before sitting on your Rollator.
- Do not stop, turn or reverse on a grade, ramp or incline.
- Do not use seat when unit is on an incline or uneven ground.
- Do not make any sharp turns.
- Replace tires when significantly worn or damaged.

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Medlogics.com Ltd.